Tesmed+

TE 780
PLUS

Manuale d'uso
Instruction manual
Mode d'emploi
Gebrauchsanweisungen
Manual de Instrucciones
Manual de Instruções

[®]
Tesmed-f

INDICE

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CONTENTS

	Page
CONTRAINDICATIONS.....	13
ELECTRO STIMULATION.....	14
ADVANTAGES OF USING THE ELECTRO STIMULATOR THERAPEUTIC / BEAUTY TREATMENT.....	15
MAINTENANCE.....	16
5 YEAR GUARANTEE.....	16
WARNINGS.....	17
Precautions / Possible side effects.....	17
DESCRIPTION AND FUNCTIONS OF THE APPLIANCE.....	18
Electrode connections.....	19
KEY TREATMENT.....	57
THERAPEUTIC TREATMENT.....	57
BEAUTY, SPORT, FITNESS TREATMENTS.....	62
FIGURE SHOWING ELECTRODE POSITIONING.....	73
TECHNICAL DATA / KIT.....	89
EMC TABLE.....	92
NORMATIVE REFERENCES / CONFORMITY DECLARATION.....	96

WARNINGS

Before using the Tesmed, read this instruction manual carefully.

Here are a few tips:

1) Before switching the Tesmed on or, in any case, before increasing intensity: always apply the electrodes to the areas to be treated and connect them via their cables to the Tesmed. If you fail to do this, the Tesmed will register an error and "E E" will appear on the display. Switch off and back on again.

2) To use a single channel: select either channel A or channel B on the display and connect the cable to the chosen channel, making sure that intensity of the channel not being used is set as 0. If not set correctly, the display registers an error. Switch off and back on again.

3) All the electrode terminals supplied are white. The colours red and black are only used in the instruction manual to help you understand better which cable plug (red or black) should be connected to the electrode shown.


4) We recommend that you increase the intensity gradually.

5) Use only Tesmed electrodes, as electrodes of a different size or composition could cause burns.

6) Always make sure that the electrodes are in proper contact with the skin, otherwise Tesmed could register an error during use.

7) Take care that the cable connector is properly inserted in the Tesmed: the arrow on the connector must face the operator, i.e., facing the front of the Tesmed (fig. 1).

8) The use of non original Tesmed electrodes and spare-parts.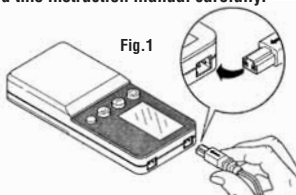

NB: WHEN SEVERAL PROGRAMS ARE REQUIRED FOR A TREATMENT, ALWAYS DO THE FIRST FOR ALL THE DAYS INDICATED BEFORE STARTING THE NEXT ONE.

CONTRAINDICATIONS

Electro stimulation is not recommended in areas with any of the following:

- inflammation, open wounds, strained or torn muscles, dermatitis, bacterial infections, eritema or skin irritation;
- muscular hernias;
- fractured bones;
- prosthesis or osteosynthesis;
- varicose veins or vascular disease in general;
- neoplasia;
- mucous membranes.

This medical device is prohibited in the following cases:

- pregnant women;
- growing children (risk of juvenile heart attack);
- patients wearing pace-makers (risk of interference with the functioning of the pace-maker);
- implanted defibrillators;
- alteration of the peripheral nerves, muscular disease and serious damage to the joints;
- patients with metal joint prosthesis and/or osteosynthesis and/or intratissue/endotissue metal;
- serious heart problems;
- patients suffering from epilepsy;
- patients with bone tumours;
- patients with serious high blood pressure;
- patients suffering from serious systematic, debilitating or dermatological disease;
- patients undergoing treatment with an HF device, as this could cause burns and damage to the stimulator itself;
- near any short wave or microwave therapy equipment, as this could cause instability in the stimulator output;
- unconscious patients or patients unable to communicate;
- people suffering from psychophysical problems that prohibit their exposure to external factors;
- in the presence of a mix of flammable anaesthetic with air, oxygen or nitrogen protoxide.

Never place the electrodes on the carotid artery and/or on a spastic muscle. Use of the TESMED for therapy purposes should be supervised and recommended by a doctor.

Interrupt application immediately in the event of discomfort and obtain medical advice. Carefully follow the points, indications, programmes and advice on use.

Electro stimulation

Electro stimulation is a technique that provokes muscular contractions very similar to voluntary contraction thanks to electrical impulses acting on the motor points of the muscles. As everyone knows, muscles contract when they receive an impulse coming from the nervous system and then return to their normal length once they are no longer stimulated thanks to their elasticity.

A motor nerve fibre splits into several ramifications before entering the muscle and ends at the motor end plate. The nerve fibre and the group of muscular fibres innervated by this make up the motor unit.

A muscular fibre responds to the electrical stimulus only when the intensity of the stimulus reaches a threshold value, which is specific to each fibre. The most obvious use of the electro stimulation of innervated muscles is in kinesiotherapy of atrophy, functional reanimation after a long period of immobility, in vasodilation, pain reducing and in stimulating the treated tissues to combat cellulite (PASSIVE EXERCISE). When treating a patient with electro stimulation, the following are crucial: the correct positioning of the electrodes, the stimulation program and the wave type.

Electrodes: the positioning of the electrodes is crucial. It must provide accurate stimulus parameters for the zone being treated since the conductivity of body tissues varies according to their structure, aqueous and ionic make-up. The electrodes must be in perfect contact with a thoroughly clean skin (also depilated, if necessary). This ensures better penetrability and permeability of the skin when receiving the stimulus to suit the application to be carried out. FOR BETTER ELECTRODE CONDUCTIVITY we recommend they be replaced when they lose adherence. They can also be cleaned with running water or wiped with a damp cloth; leave to dry before using them again for a few more applications. We recommend using Tesmed electrodes exclusively, as electrodes of different sizes and composition could cause burns. NB: With electro stimulation, the current density diminishes as the tissue consistency increases.

Tiredness: excessive contraction/relaxing of the muscles due to electrical stimulus often results in a drop in the relaxation of the neurotransmitters, thus resulting in muscle fatigue. The scientific explanation is as follows: the stimulus tends to provoke the recruitment of large diameter motor neurones (these have a low threshold), thus exciting the more consistent and faster muscle fibres (FIBRES 2 or white fibres). These tire more easily, but they use less effort than FIBRES 1 or red fibres. **In other words, the higher the frequency of stimulus, the more tired the muscle becomes.** For this reason, it is important to include relaxation programs and get medical advice. Having said this however, Tesmed's sports programs already contain a relaxation phase. During treatments using the electro stimulator, every increase in intensity must be done gradually. If there is EXCESSIVE contraction, you must reduce the intensity. In fact, stimulation must not provoke muscular tetany (i.e. long continuous contractions). There are no standard rules for the number of contractions and the duration and frequency of the sessions. These must be set to suit each person and the condition of the tissues to be treated. When treating areas affected by CELLULITE, you should normally start the treatment by gradually raising the intensity until the patient starts to feel a slight tingling, but without creating excessive contraction of the muscles or an unpleasant sensation of localised heat.

To sum up: the vasomotor and metabolic effects induced by the passage of an electrical current considerably increase the consistency and quality of tissues.

The best level of stimulus is when the patient perceives the current passing through the tissue; this should provoke a light initial contraction of the muscles (a slight trembling), but never too intense.

There are two important advantages here:

- 1) you avoid the risk of muscular over-contraction and the accumulation of lactic acid;
- 2) the frequency of muscular contractions may be much higher than with traditional gym, to get best results faster.

Advantages of using the electro stimulator THERAPEUTIC TREATMENT

In rehab: post trauma and post surgery treatment of muscular atrophy; recovery of muscular strength; healing of contracture; motor recovery.

In welfare: pain relief; increased endorphins; maintenance of muscular tone. Muscular damage due to electro stimulation is most unlikely, as long as the intensity is increased gradually. Any interference in the menstrual cycle (e.g. early, late, heavy or light periods) are extremely subjective and variable. We recommend, therefore, to avoid treatments in the abdominal area during the actual period and the days immediately before and after this. To date, no side effects have been noted concerning the use of electro stimulation during lactation. However, we recommend not treating the upper body area during this time. We strongly advise against treating any parts of the body with serious skin problems.

Intended use: the Tesmed medical device is intended to be used exclusively for the following treatment:

- **Excitomotors** = contraction of a muscle or a group of muscle fibres
- **Vasomotors** = vasodilation effect
- **Pain killer** = reduction of pain

See the relevant program in this instruction manual and get medical advice for each problem.

More specifically, the equipment must not be used for:

- Electrodiagnosis,
- Vehiculation (introduction of medicinal ions),
- Experimental use,
- Operations not described in the examples and instructions provided in this manual.

Advantages of using the electro stimulator BEAUTY TREATMENT

In beauty: improvement of capillary micro circulation with more oxygen supplied to tissues, thus producing better fat mobilisation; toning up and firming up of tissues; draining of excess liquids; an increase in metabolism and cellular activities; a reduction of localised fatty tissues and cellulite.

In sports: isolation of the muscular group being treated; saving of physical and psychological energy; recruitment of a larger number of muscular fibres than is the case with voluntary contraction; training of under-used muscular fibres during normal work-out sessions; shorter recovery periods; protection of the body's system of tendons and muscles; improved blood circulation.

Results: all results in the aesthetics field are naturally subjective. We can, in any case, sustain that in the case of TONE UP, good results can be seen after just 15 days with 5-7 regular and constant weekly sessions; cellulite treatment will require 30-40

days. Better and quicker results may be obtained if treatments are added to suitable physical activity and a correct lifestyle.

Warnings:

- The Tesmed must not be used for purposes other than those indicated.
- **We recommend using original electrodes only to guarantee top performance, as these have been developed specifically for the Tesmed. Non original electrodes could burn the skin.**
- The Tesmed must be kept out of the reach of children.
- Electrocardiogram monitoring equipment may be disturbed by the current used for the electro stimulator.
- We do not recommend using the Tesmed on children under the age of 14 or the elderly over 75.
- The electrodes must not be placed less than 10 cm from the eyeballs.
- Never place the electrodes on the carotid sinus or on spastic muscles.
- Do not use the equipment on patients with special health problems without first obtaining medical advice from their doctor.

Maintenance

Only original spare-parts and products must be used for maintenance operations. To keep the Tesmed in perfect efficiency, you must observe the following maintenance intervals:

- A)** Regularly (monthly if used regularly, every fortnight if used intensively): check the general state of the electrode cables. There should be no signs of rusting.
- B)** Monthly: clean the body of the machine with a soft cloth dipped in alcohol (NB: switch the machine off and remove the batteries).
- C)** Regularly (at the first signs of wear): replace the electroconducting plates. The manufacturer cannot be held liable for any provision of the warranty if the above recommendations are ignored. The manufacturer guarantees this machine will have a useful life of 5 years with correct use and maintenance.

Guarantee

tesmed.com

Warnings

- All therapies must be indicated by expert physicians.
- The technical data for each program and the relevant paragraphs in this instruction manual must be read.
- When using the T esmed for electrotherapy of muscles in the head, apply the electrodes at least 10 cm from the eyes.
- Each treatment must be started with zero current emission (minimum channel output); gradually increase/decrease the intensity within tolerable limits.
- Use of the max intensity does not mean better performance or results.
- The use of electricity conducting plates of a smaller diameter than that recommended is strictly prohibited.
- Never leave or abandon liquid on the equipment.
- Always protect the equipment from dust and humidity.
- Should the T esmed malfunction, do not try to open or repair it yourself, but send the product in its original packaging to GLOBAL TRADE S.r.l., including a written description of the problem inside.

Precautions

- Do not tamper with the appliance. Do not open it.
- Keep the electro stimulator away from heat, dust, water sources and humid places and avoid direct exposure to the sun.
- Avoid violent knocks that might damage the appliance.
- Clean the electro stimulator using a soft cloth only.

Possible side effects


Isolated cases of cutaneous irritation may appear in individuals with particular skin sensitivity. Should there be an allergic reaction to the gel used with the electrodes, interrupt the treatment immediately and get expert medical advice.

CAUTION

- FOR PROGRAM WHERE ONLY ONE CHANNEL IS USED, ALWAYS KEEP THE OTHER CHANNEL NOT IN USE AT 0 INTENSITY, OTHERWISE THE APPLIANCE WILL REGISTER AN ERROR.
- FOR PROGRAMS WHERE THE USE OF ONLY ONE CHANNEL IS INDICATED, YOU MAY STILL USE THE OTHER CHANNEL (APPLYING THE ELECTRODES AS SHOWN IN THE PHOTO), BUT THESE MUST BE APPLIED TO THE OPPOSITE SIDE OF THE BODY OR ON THE OPPOSITE LIMB.

**WE RECOMMEND INCREASING INTENSITY GRADUALLY
IN ORDER TO AVOID SUDDEN CONTRACTIONS**

Description of the appliance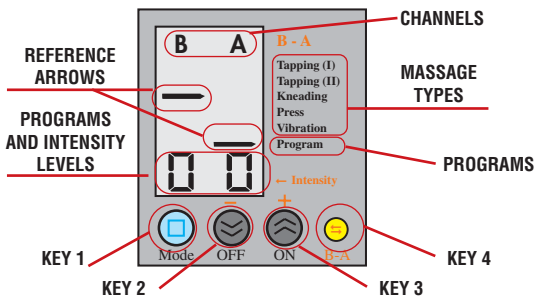

Functions of the appliance

SWITCH ON: PRESS KEY 3.

SWITCH OFF: KEEP KEY 2 PRESSED FOR ABOUT 3 SECONDS. THE APPLIANCE WILL TURN OFF REGARDLESS OF THE CURRENT STATUS. ON PRESSING KEY 2 THE TENSMED IS IMMEDIATELY SWITCHED OFF IF THE INTENSITY = 0 (WITHOUT HAVING TO WAIT 3 SECONDS).



CHANNEL SELECTION: PRESS KEY 4 TO SELECT A SINGLE CHANNEL OR BOTH CHANNELS A-B.

PROGRAM SELECTION: PRESS KEY 1 TO SELECT FIRSTLY THE INDIVIDUAL TYPES OF MESSAGE (TAPPING (I), TAPPING (II), KNEADING, PRESS OR VIBRATION) (AT LEAST ONE OF THE FLASHING ARROWS WILL GO NEXT TO THE TYPE OF MESSAGE DESIRED), FOLLOWED BY PROGRAMMES 1/45, THESE WILL START TO FLASH WHEN ONE OF THE TWO ARROWS IS POINTING TO PROGRAM. KEEP KEY 1, PRESSED TO SPEED UP THE PROGRAM SELECTION; WHEN YOU GET CLOSE TO THE PROGRAM YOU WANT, WE RECOMMEND TAPPING ON KEY 1 TO ADVANCE ONE PROGRAM AT A TIME. PRESS KEY 3 TO CONFIRM THE PROGRAM, AS INDICATED IN "INCREASE INTENSITY" BELOW.

INCREASE INTENSITY: WHEN THE NUMBER OF THE PROGRAM YOU WANT TO USE STARTS FLASHING, PRESS KEY 3 TO CONFIRM THE PROGRAM AND THE APPLIANCE WILL AUTOMATICALLY START WORKING AT THE LOWEST INTENSITY LEVEL OF 0.5 (NUMBER 1 FLASHES). EVERY TIME YOU PRESS KEY 3 THE INTENSITY INCREASES BY 0.5 (ONE STEP AT A TIME, MAX 18 STEPS). PRESS KEY 2 TO LOWER THE INTENSITY. WHEN YOU PRESS KEY 2 WHEN THE INTENSITY = 0, THE APPLIANCE WILL SWITCH OFF. THE INTENSITY VARIES BY ITSELF DURING THE 45 PRE-SELECTED PROGRAMMES EVERY TIME THE TYPE OF MESSAGE IS CHANGED. YOU CAN, IN ANY CASE, CHANGE THE INTENSITY, BUT THE APPLICATION TIME OF 20 MINUTES CANNOT BE MODIFIED.

Electrode connections

FOR EASY USE, ALWAYS CONNECT THE GREEN CABLE TO TESMED CHANNEL A AND THE ORANGE CABLE TO CHANNEL B (Figure 1). THEN CONNECT THE PLUGS, BEARING IN MIND THE POLARITY, RED/BLACK (Figure 2).

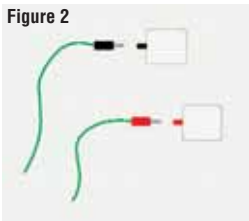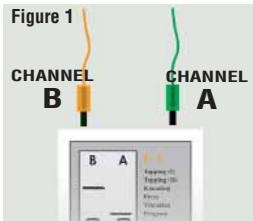PLACE THE ELECTRODES (AS IN THE FIGURE IN THE INSTRUCTIONS MANUAL) SHOWING THE CHOSEN TREATMENT AND CONNECT THE PLUGS.

EXAMPLE: RED PLUG ON THE CANALE A CABLE (GREEN) SHOULD BE CONNECTED TO AN ELECTRODE MARKED WITH A RED RECTANGLE, PLACED AS SHOWN IN THE RELEVANT FIGURE FOR THE CHOSEN TREATMENT. REPEAT THE SAME OPERATION WITH ALL THE OTHER PLUGS.

USE OF INDIVIDUAL TYPES OF MESSAGE: TESMED LETS YOU PERFORM INDIVIDUAL MESSAGES AT WILL (the timer is programmed for 20 minutes):

TAPPING (I) = LIGHT PERCUSSIVE TAPPING MESSAGE

TAPPING (II) = QUICK TAPPING MESSAGE

KNEADING = KNEADING MESSAGE

PRESS = MESSAGE WITH PRESSURE

VIBRATION = MESSAGE WITH VIBRATION

MORE POTENCY: FOR MORE DEMANDING AND EXPERIMENT PERSONS, 2 ELECTRODES CAN BE USED PER CHANNEL TO DOUBLE THE POTENCY IN THE ZONE WHERE THE ELECTRODE IS APPLIED. IN THIS CASE, WE RECOMMENDED USING ONE RED PLUG AND ONE BLACK ONE PER CHANNEL, POSITIVE AND NEGATIVE, COVERING THE OTHER TWO WITH THE COVERS INCLUDED IN THE KIT. THIS SOLUTION IS NOT RECOMMENDABLE FOR INEXPERIMENT USERS. WE RECOMMEND INCREASING THE INTENSITY WITH CAUTION, EVEN IF YOU ARE AN EXPERT, AS THE INTENSITY IS ALWAYS DOUBLED WITH TWO ELECTRODES.

TIMER: ALL PROGRAMS AND ALL MODES HAVE A FIXED MAXIMUM FUNCTIONING TIME OF 20 MINUTES. THE SYSTEM AUTOMATICALLY RETURNS TO "0" AFTER 20 MINUTES AND, THEN AFTER A FURTHER 4 MINUTES, IT WILL AUTOMATICALLY SWITCH OFF. IF YOU START WITH ANY PROGRAM BELONGING TO CHANNEL A AND THEN, AFTER 10 MINUTES, CHANNEL B, IS ENABLED BY ANY PROGRAM, THE TIMERS CONTROLLING THE TWO CHANNELS WILL WORK INDEPENDENTLY AND SO, ONCE THE 20 MINUTES ARE UP FOR

CHANNEL A, ITS INTENSITY WILL DROP TO "0" AND THEN, ONCE THE 20 MINUTES ARE UP FOR CHANNEL B, THIS WILL GO TO "0" AND THE EQUIPMENT WILL AUTOMATICALLY SWITCH OFF. IF THE SYSTEM IS SWITCHED ON, BUT IS NOT ACTIVATED, THE APPLIANCE WILL SWITCH OFF AFTER 4 MINUTES.

INTENSITY CONTROL

THE INTENSITY CONTROL SCALE IS SPLIT INTO THE FOLLOWING 18 LEVELS:

- 1 flashing = intensity level 0,5
- 1 steady = intensity level 1
- 2 flashing = intensity level 1,5
- 2 steady = intensity level 2
- 3 flashing = intensity level 2,5
- 3 steady = intensity level 3
- 4 flashing = intensity level 3,5
- 4 steady = intensity level 4
- 5 flashing = intensity level 4,5
- 5 steady = intensity level 5
- 6 flashing = intensity level 5,5
- 6 steady = intensity level 6
- 7 flashing = intensity level 6,5
- 7 steady = intensity level 7
- 8 flashing = intensity level 7,5
- 8 steady = intensity level 8
- 9 flashing = intensity level 8,5
- 9 steady = intensity level 9 (maximum intensity)

Do not use TAPPING (I) or TAPPING (II) on one channel at the same time as KNEADING, PRESS or VIBRATION on the other channel.

ERROR FUNCTION (DETACHED ELECTRODE): IF AN ELECTRODE IS DETACHED, THE APPLIANCE REGISTERS AN ERROR. ONLY "E E" (FLASHING) WILL APPEAR ON THE DISPLAY TO INDICATE AN ANOMALY AND ALL OUTPUT SUPPLY WILL BE CUT. TO RESET THE SYSTEM, SWITCH IT OFF, CONNECT THE ELECTRODE AND SWITCH BACK ON. OTHERWISE, THE SYSTEM WILL AUTOMATICALLY SWITCH OFF AFTER 4 MINUTES. **N.B.** WHEN USING THE DOUBLE CABLE FOR 8 ELECTRODES, THIS FUNCTION WILL ONLY WORK IF BOTH THE 2 ELECTRODES OF THE SAME POLARITY ARE DETACHED (2 BLACK ONES OR 2 RED ONES). THIS FUNCTION IS INFLUENCED BY THE DIFFERENCE IN RESISTANCE IN EACH INDIVIDUAL AND SO THE LOW INTENSITY DETECTION TIME MAY BE INACTIVE.

DEMO FUNCTION: THIS FUNCTION IS FOR USING THE APPLIANCE WITHOUT ANY CURRENT AND WITHOUT ANY ERROR SIGNALS. TO USE THIS FUNCTION, SWITCH THE APPLIANCE OFF AND THEN PRESS KEY 4 FOR 5 SECONDS. YOU ARE IN DEMO MODE WHEN INTENSITY = 8 8 AND CHANNELS A-B APPEAR WITH THE TWO FLASHING ARROWS POINTING TO TAPPING 1. YOU CAN VIEW ANY PARAMETER FOR A CHOSEN PROGRAM WITHOUT ACTIVATING THE TIMER FUNCTION OR THE OUTPUT SIGNAL. **NB:** DEMO FUNCTIONS DO NOT ALWAYS MATCH THE REAL ONES. TO QUIT DEMO MODE, SWITCH THE SYSTEM OFF AND THEN SWITCH BACK ON.

RESET FUNCTION: IN CASE OF MALFUNCTIONING RESET THE TESMED AFTER ANY MALFUNCTION BY SIMPLY REMOVING AND REINSERTING THE BATTERIES.

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Guide and manufacturer's statement - electromagnetic immunity

The TE 780 PLUS electro stimulator is designed to work in the electromagnetic environment described below. The customer or user of the TE 780 PLUS electro stimulator must guarantee that it used in such an environment.

Immunity test	IEC 60601 test level	Conformity level	Electromagnetic environment - guide
Electrostatic discharge (ESD) IEC 61000-4-2	± 6kV contact ± 8kV in air	± 6kV contact ± 8kV in air	Floors must be wooden, concrete or tiled. If floors are covered with a synthetic material, the relative humidity should be at least 30%.
Transiting / fast electric trains IEC 61000-4-4	± 2kV for power lines ± 1kV for input/output lines	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery.
Overvoltage IEC 61000-4-5	± 1kV differential mode ± 2kV common mode	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery
Voltage drops, brief cuts and variations in voltage on supply input lines IEC 61000-4-11	5% U_T (>95% drop in U_T) for 0.5 cycles 40% U_T (60% drop in U_T) for 5 cycles 70% U_T (30% drop in U_T) for 25 cycles <5% U_T (>95% drop in U_T) for 5 seconds	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery
Magnetic field at mains frequency (50/60 Hz) IEC 61000-4-8	3 A/m	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery

NOTE: U_T is the AC mains voltage rate prior to application of the test level.

Recommended separation distances between portable and mobile radio frequency equipment and the TE 780 PLUS electro stimulator

The TE 780 PLUS electro stimulator is designed to work in an electromagnetic environment where irradiated RF interference levels are kept under control. The customer or the user of the TE 780 PLUS electro stimulator may help avoiding electromagnetic interference by guaranteeing a min distance between mobile and portable RF communication devices (transmitters) and the TE 780 PLUS electro stimulator, as recommended below and to suit the max output power of the radio communication devices.

Max output power of the specified transmitter (W)	Separation distance at the transmitter frequency (m)		
	150kHz - 80MHz	80MHz - 800MHz	800MHz - 2,5GHz
	$1,17\sqrt{P}$	$1,17\sqrt{P}$	$1,17\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,17	1,17	2,33
10	3,7	3,7	7,37
100	11,7	11,7	23,3

Transmitters with a max output power not indicated above: the recommended separation distance in metres (m) can be calculated using the applicable equation for the transmitter frequency, where P is the max nominal output power of the transmitter in Watts (W) as indicated by the manufacturer of the transmitter and d is the recommended separation distance in metres (m). **NOTE 1:** At 80 MHz and 800 MHz, the highest frequency interval is applied.

NOTE 2: These guidelines may not apply to all situations. Electromagnetic propagation is influenced by the absorption and reflection of structures, objects and people.

Guide and manufacturer's statement - electromagnetic immunity

The TE 780 PLUS electro stimulator is designed to work in the electromagnetic environment described below. The customer or user of the TE 780 PLUS electro stimulator must guarantee that it used in such an environment.

Immunity test	IEC 60601 test level	Conformity level	Electromagnetic environment - guide
RF carried IEC 61000-4-6	3Vrms 150kHz - 80MHz	Not applicable	Portable and mobile RF communication devices should not be used any closer to any part of the TE 780 PLUS electro stimulator, including the cables, than the recommended separation distance calculated using the applicable equation for the transmitter frequency. Recommended separation distance: $d = 1,17 \sqrt{P}$ $d = 1,17 \sqrt{P} \quad 80\text{MHz} - 800\text{MHz}$ $d = 2,33 \sqrt{P} \quad 800\text{MHz} - 2,5\text{GHz}$ where P is the max nominal output power of the transmitter in Watts (W) as indicated by the manufacturer of the transmitter and d is the recommended separation distance in metres (m). The intensity of fixed RF transmitters (as established by an electromagnetic survey ^a of the site) could be less than the conformity level in each frequency interval ^b . There may be interference near devices marked: 
RF irradiated IEC 61000-4-3	3V/m 80MHz - 2,5GHz	3V/m 80MHz - 2,5GHz	

NOTA 1: At 80 MHz and 800 MHz, the highest frequency interval is applied.

NOTA 2: These guidelines may not apply to all situations. Electromagnetic propagation is influenced by the absorption and reflection of structures, objects and people.

A) Field intensity for fixed transmitters, such as radio phone stations (mobile phones and cordless phones) and walkie-talkies, radio ham equipment, AM and FM radio equipment and TV transmitters cannot be foreseen in theory or with precision. To assess an electromagnetic field causes by fixed RF transmitters, you need to carry out an electromagnetic survey of the site. If the field intensity found in the place where the TE 780 PLUS electro stimulator is to be used exceed the applicable conformity level above, you will need to observe the normal working of the TE 780 PLUS electro stimulator. If there are anomalies, extra measures will be needed, such as a different position or direction of the TE 780 PLUS electro stimulator.

B) The field intensity in the frequency range 150 kHz to 80 MHz should be less than 3V/m.

Guide and manufacturer's statement - electromagnetic emissions

The TE 780 PLUS electro stimulator is designed to work in the electromagnetic environment described below.

The customer or user of the TE 780 PLUS electro stimulator must guarantee that it used in such an environment.

Emission test	Conformity level	Electromagnetic environment - guide
RF emissions CISPR 11	Group 1	The TE 780 PLUS electro stimulator only uses RF energy for its internal functioning. Therefore, its RF emission is very low and is unlikely to cause any interference with any electronic equipment nearby.
RF emissions CISPR 11	Class B	—————
Harmonic emissions IEC 61000-3-2	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery.
Voltage fluctuation emissions/flicker IEC 61000-3-3	Not applicable	The TE 780 PLUS electro stimulator is a device with DC power supply from an incorporated battery.

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GB **NORMATIVE REFERENCES** The appliance called TESMED TE 780 PLUS is designed and built to comply with the following laws and standards: •EEC Directive 93/42 dated 14th June 1993 concerning medical devices, implemented in Italy by Legislative Decree n° 46 dated 24/02/97 and subsequent amendments, •UNI IEC EN ISO 14971 standards, •UNI EN 980 standards 2003 edition, •IEC EN 60601-1, •IEC EN 60601-1-2, •IEC EN 60601-2-10, •IEC EN 55011, •IEC EN 61000-4-2, •IEC EN 61000-4-3.

Conformity statement GLOBAL TRADE S.r.l., with registered offices in Via delle Betulle 25, Treviolo (BG) Italy and production site in Via V Alpini 23/a, Bagnatica (BG) Italy, hereby declares that the medical device called "TESMED TE 780 PLUS", in its applications as a medical device (according to the intended use, the indications and contraindications), risk class IIa as per rule 9 of Enclosure IX to EEC Directive 93/42 (implemented in Italy by Legislative Decree n° 46 dated 24th February and subsequent amendments) •complies with the essential requirements and provisions of EEC Directive 93/42, as per Technical Bundle n° FT 001 GT; •is manufactured in accordance with the Quality System that satisfies the requirements of Enclosure V of the aforementioned Legislative Decree, as per Certificate n° MED 24046 issued by CERMET - Notified Body n° 0476, Via Cadriano 23, 40057 Cadriano di Granarolo Emilia (BO) Italy.

The Legal Representative

Tesmed accessoires and consumables, and new products can be obtained from your local chemist.

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All rights reserved. Any reproduction,